Otto Bock®

QUALITY FOR LIFE

CE

z50

(D)	Bedienungsanleitung.....	1
(GB USA)	Instructions for Use.....	29

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Instructions for Use – z50

Contents	Page
1 General Information	31
2 Declaration of Conformity	31
3 Safety Instructions	32
3.1 Warning Symbols	32
3.2 General Safety Instructions	32
4 Range of Application	35
5 Service	35
6 Delivery	36
7 Putting into Operation	36
7.1 Assembly of the z50 Wheel.....	36
7.1.1 Assembly of the Manual Wheelchair Wheel	38
7.1.2 Disassembly of the z50 Wheel	38
7.1.3 Disassembly of the Manual Wheelchair Wheel.....	38
7.2 Switching On.....	39
7.3 Drive Mode	39
7.3.1 Push Mode	40
7.4 Drive-away Lock	41
7.6 Battery Indicator	41
7.7 Battery Charging	42
7.7.1 Charging Process.....	42
7.7.2 Error Indicator of the Battery Charger.....	43
7.8 Battery.....	44
7.8.1 Removing the Battery.....	44
7.8.2 Installing the Battery	45
7.9 Fuse	45
7.10 Removing the Control Panel.....	46
7.11 Anti-tippers	46
8 Transport and Storage	46
9 Status and Error Analysis	47
10 Options	48
10.1 Control Panel.....	48
10.2 Swinging Anti-tippers	49
10.2.1 Removing the Swinging Anti-tippers	50
10.2.2 Attaching the Swinging Anti-tippers	50
10.2.3 Joystick Accessories	51
11 Maintenance and Cleaning	52
11.1 Maintenance.....	52
11.2 Cleaning.....	52

12	Disposal	53
13	Technical Data of the z50	53
13.1	Weight Indications	53
13.2	Electrical Installation	53
13.3	Battery Charger.....	53
13.4	Driving Data	54
14	Terms of Extended Warranty.....	54
14.1	Subject of the Extended Warranty.....	54
14.2	Scope of the Extended Warranty	54
14.3	Conditions for Laying Claims	54
14.4	Exclusion of the Extended Warranty	54
14.5	Liability	55
14.6	Incidental Provisions	55

1 General Information

Preface

Congratulations on your choice of the z50 power add-on drive, an innovative and functional state-of-the-art assistance device.

This manual will show you how you can quickly and easily make use of all functions. Should you have further questions, please contact your specialized dealer.

The model as described in these Instructions for Use is subject to technical changes without notice.

Please observe the maintenance and cleaning instructions at the end of this manual.

Before mounting and putting into operation the power add-on drive, please read these Instructions for Use thoroughly. Please also read the Instructions for Use for your wheelchair, even if you are already familiar with its functions and handling.

These Instructions for Use refer to the adaptation as well as the drive unit with its components. The subsequent sections contain further information for an optimal use of the power add-on drive as well as a first support if any service is required. Both, user and attendant, should read the "Safety Instructions" section with particular attention.

2 Declaration of Conformity

Otto Bock HealthCare GmbH as manufacturer with sole responsibility declares that the z50 power add-on drive conforms to the requirements of the European Directive for Medical Products 93/42/EEC.

Development, construction and design of this product fully meet the requirements regarding technical safety laid down in the following standards:

- **DIN EN 12182 Technical aids for disabled persons "General requirements and test methods"**
- **DIN EN 12184 Electrically powered wheelchairs, scooters and their chargers "Requirements and test methods"**

3 Safety Instructions

3.1 Warning Symbols

DANGER

Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury or technical damages.

NOTE

Information for operating the product.

NOTE

Please read the Instructions for Use first!

NOTE

Information regarding environment protection.

3.2 General Safety Instructions

DANGER

Operating errors can lead to accidents, resulting in a risk of injury to you. Therefore, please make sure you are familiar with the product and how to operate it before using it. Practice using the product on a level and clear terrain.

DANGER

Uncontrolled operation of the control can lead to accidents, resulting in a risk of injury to you. Before getting into or out of the wheelchair with power add-on drive, switch off the control of the add-on drive. When using lifting platforms, elevators or public means of transportation, also switch off the control and always engage the wheel locks.

DANGER

Defective brakes/wheel locks can lead to accidents, resulting in a risk of injury to you. Check the brakes/wheel locks of your wheelchair with power add-on drive each time before driving.

DANGER

Driving on inclines and slopes is dangerous. Improper use can lead to accidents, resulting in a risk of injury to you. Do not lean too far backward when driving uphill and do not lean too far forward when driving downhill. Avoid tilting and do not lean sideways out of the wheelchair.

DANGER

Get to know how the z50 power add-on drive reacts when the center of gravity shifts; for example on slopes or inclines or when clearing obstacles, like steps and curbs. This should be done only with assistance from another person.

DANGER

Use caution near flammables or fire, especially when holding lit cigarettes. The back upholstery and seat cushion may catch fire. This can result in serious burns.

DANGER

When starting or accelerating, there is increased risk of tipping. Use the z50 power add-on drive exclusively with approved anti-tippers. Mount the corresponding anti-tippers on both sides.

DANGER

When driving on unsuitable surfaces, there is an increased risk of accident. This results in a risk of injury to you. Avoid driving on icy surfaces and on grounds such as gravel, pebbles, shingle or scree.

DANGER

Driving up to an obstacle in the wrong way can cause you to fall, resulting in a risk of injury to you. Always approach obstacles (such as steps or curbs) at a right angle. Please bear in mind that the anti-tippers restrict the obstacle height that can be cleared.

DANGER

Never grip into the wheel spokes or between the wheels and the frame or at the brake lever handles when driving with the power add-on drive. This could result in painful injury. Lay your forearms on the armrests when driving with the power add-on drive. Operate the brake lever handles only when the wheelchair is standing still and only use the ball of the thumb for operation. Always keep your fingers away from the spokes.

DANGER

Z50 wheels that have not been properly engaged may come loose while driving. This can lead to a fall and to serious injuries. Each time before starting, make sure that the z50 wheels are firmly engaged.

DANGER

The driving characteristics of the z50 power add-on drive can be influenced by electromagnetic fields. Switch off all mobile devices (such as mobile phones) while driving.

DANGER

Explosive gases can develop while the batteries are charging. For this reason, provide for sufficient ventilation when using the battery charger in closed rooms. Do not smoke near the battery charger or while handling the batteries. Avoid fire and sparks.

CAUTION

When using lifting platforms, be sure that the wheelchair with z50 power add-on drive is standing centrally on the lifting platform area and that none of the wheelchair's components, such as the anti-tippers, is protruding into the danger area. Non-observance of this instruction can cause damage to the wheelchair or to the power add-on drive.

CAUTION

Total discharge will destroy the battery. Do not cover longer distances with the battery indicator showing reserve capacity. Charge the battery of the power add-on drive at the latest when 2 LEDs are still lit.

CAUTION

If not restricted by the manual wheelchair, the maximal load capacity with mounted z50 power add-on drive is 125 kg.

CAUTION

The effectiveness of the wheel lock and the general driving ease are dependent on adequate tire air pressure and tread depth. Therefore, ensure the tires have sufficient tread depth and check that the pneumatic tires are inflated to the correct tire pressure (see maintenance instructions).

CAUTION

The z50 power add-on drive may be used within a temperature range from -25 °C to +50 °C.

CAUTION

Water will destroy the drive unit. Never use a water hose or high-pressure cleaning apparatus for cleaning the power add-on drive. Water could enter the drive unit and irreparably damage the electronic components.

CAUTION

Overheating will destroy the battery charger. When charging batteries, make sure that the cooling ribs on the back side of the connected battery charger are not covered.

NOTE

It is possible that the z50 power add-on drive itself generates electromagnetic fields that might cause interference in other devices. Therefore, switch off the control whenever you do not need it.

NOTE

To allow the complete charging current to be fed into the battery, the control is to be switched off during the charging process.

NOTE

For safety reasons, we recommend attaching a pelvic belt, which is available as an accessory.

NOTE

When the z50 power add-on drive is not used over extended periods of time or for dispatching the add-on drive, the battery plug should be removed to avoid standby current of the control!

4 Range of Application

The z50 power add-on drive in combination with a manual wheelchair is designed for individual mobility. It is intended solely for people who are unable to walk or who have a walking impediment. The z50 can also be optionally operated by attendants.

The power add-on drive is delivered in combination with the following Otto Bock wheelchairs:

- Avantgarde T
- Avantgarde VR
- Start Basic
- Start Multi
- Start Plus

Upon request, our custom fabrication department will check the possibility of mounting the z50 to other Otto Bock wheelchair models or to wheelchairs of other manufacturers.

5 Service

Service and repairs on the z50 power add-on drive may only be carried out by authorized dealers.

Your authorized dealer:

6 Delivery

The z50 power add-on drive is ready for use upon delivery by the dealer. The basic settings will be based on the indications on the order form and will be adapted to your requirements on site by the specialized dealer as far as necessary. Before using the z50 for the first time, all components of the standard equipment (Fig. 1) must be checked for completeness and possible damages.

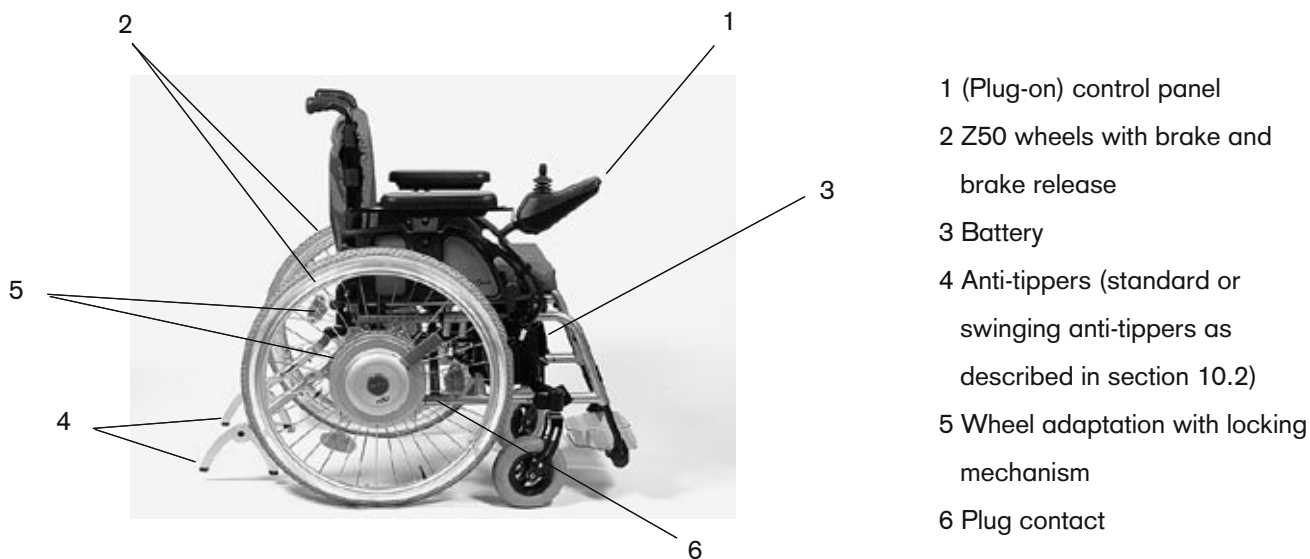
Fig. 1 Components of the z50

- 1 (Plug-on) control panel
- 2 Z50 wheels with brake and brake release
- 3 Battery
- 4 Anti-tippers (standard or swinging anti-tippers as described in section 10.2)
- 5 Wheel adaptation with locking mechanism
- 6 Plug contact

The manual wheelchair wheels with mounting plate are also included.

The functions of the individual components must be checked according to the instructions in the section "Putting into Operation". Should any problems arise, please refer to the section "Status and Error Analysis".

7 Putting into Operation

DANGER

Z50 wheels that have not been properly engaged may come loose while driving. This can lead to a fall and to serious injuries.

Each time before starting, make sure that the z50 wheels are firmly engaged.

Before the first use, the correct assembly of the z50 wheels must be checked.

7.1 Assembly of the z50 Wheel

CAUTION

To prevent unintentional release of the locking mechanism, the locking lever must be pressed downwards after folding in the z50 wheel. Correct locking is indicated by engaging the red safety pin.

CAUTION

After mounting of the z50 wheel, the correct function of the wheel lock must be checked (refer to the Instructions for Use of the manual wheelchair).

The z50 wheels are attached to the adapter plates and locked by means of a hinge lock. Make sure, that the right z50 wheel is mounted to the right side of the wheelchair and the left z50 wheel to the left side. "R" and "L" labels on the inner side of the respective motor distinguish the z50 wheels from each other. Moreover, the brake lever handles must point in driving direction.

Fig. 2 Positioning the z50 wheel

Fig. 3 Plug connection

Motor connection is provided by a self-contacting plug connection which is established as soon as the hinge lock engages.

The mounting plate with the receiver hook for the attachment bolt and the motor plug is attached to the z50 wheel. Position the system beside the wheelchair (Fig. 2).

Fig. 4 Attach the attachment bolt into the receiver hook

Fig. 5 Assembly of the z50 wheels

The receiver hook for the attachment bolt is attached to the attachment bolt of the adapter plate (Fig. 4).

Now fold the z50 wheel towards the wheelchair until it is in vertical position and the locking lever engages in the bushing (Fig. 4).

Fig. 6 Locking lever

NOTE

If it is not possible to completely fold in the z50 wheel, this is because the locking lever has already been activated. In this case, first pull the safety pin to release the lock.

Now press the locking lever down to engage the red safety pin. In this position, the locking mechanism is protected against unintentional release.

7.1.1 Assembly of the Manual Wheelchair Wheel

DANGER

For proper functioning of the wheel lock it must be ensured that the z50 wheel and the manual wheel have the same diameter and tread. Furthermore it is to be checked, if the adapter plates of the z50 wheel and of the manual wheel are adjusted to the same height.

DANGER

After mounting of the manual wheel, the function of the wheel lock must be checked (refer to the Instructions for Use for your manual wheelchair).

The assembly of the manual wheelchair wheel takes place in the same way as the assembly of the z50 wheel. A mounting plate with quick-release axle housing on the manual wheelchair wheel is attached to the hinge lock and locked. Again, it must be ensured that the red safety pin engages.

7.1.2 Disassembly of the z50 Wheel

To disassemble the z50 wheel, first the locking lever (black) must be pressed down slightly. Now the red safety pin can be pulled.

Fig. 7 Pull the safety pin

Fig. 8 Disassembling the z50 wheel

This will release the locking lever, which can now be pulled up.

Slightly lifting the wheelchair will allow the z50 wheel to be tilted outwards, and the wheel can be removed from the hinge lock.

7.1.3 Disassembly of the Manual Wheelchair Wheel


The manual wheelchair wheel can be disassembled in the same way as the z50 wheel. It is also possible, however, to leave the adapter plate in the mounting plate and remove and reattach the wheel like with the manual wheelchair using the quick-release axle system.

For more information on the quick-release axle system, please refer to the Instructions for Use of your manual wheelchair.

7.2 Switching On

The buttons and joystick of the control panel (Fig. 9) are used to operate the z50 power add-on drive. The control panel is designed as integral control and thus contains the complete electronic control system for the wheelchair. Press button 1 to switch on the control. The indicator 8 shows the remaining battery capacity. The indicator 7 will show 1-5 lit LEDs depending on the selected speed level. When the speed level indicator flashes or the LEDs continually light up one after another, this means a system error has occurred. When the control unit is not used for 10 minutes, it is switched off automatically. The control can be switched off at any time; if this occurs whilst driving the wheelchair will be braked immediately.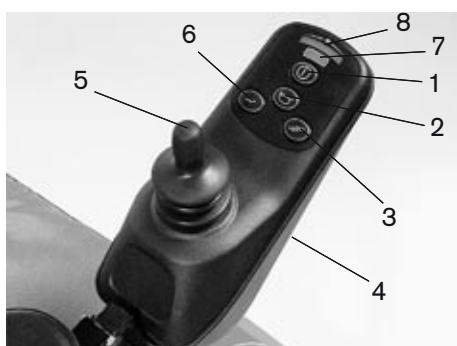

- 1 ON/OFF button
- 2 Horn
- 3 Increase speed level
- 4 Charging receptacle (bottom side)
- 5 Joystick
- 6 Reduce speed level
- 7 Speed level and system error indicator
- 8 Battery indicator

Fig. 9 Control panel

7.3 Drive Mode

Before starting your wheelchair please read section 3 "Safety Instructions"!

CAUTION

Total discharge will destroy the battery.
Do not cover longer distances with the battery indicator showing reserve capacity.
Charge the battery of the power add-on drive at the latest when 2 LEDs are still lit.

DANGER

Uncontrolled operation of the control can lead to accidents, resulting in a risk of injury to you.
Before getting into or out of the wheelchair with power add-on drive, switch off the control of the add-on drive.
When using lifting platforms, elevators or public means of transportation, also switch off the control and always engage the wheel locks.

When switching on the wheelchair control, it will automatically be set at the speed level that was last used prior to switching the system off. Button 3 is used to increase the speed level and button 6 to reduce it. For driving use the joystick (5). The further the joystick is moved away from the mid-position, the faster the wheelchair with z50 power add-on drive will drive in this direction. The maximum speed with full deflection of the joystick is dependent on the selected speed level. The characteristics for speed, acceleration and deceleration can be adapted by the dealer to meet the individual user's requirements.

As soon as the joystick is released, the wheelchair's brake function is automatically initiated by the control until it stops moving. When standing still, the mechanical brakes are automatically active so the wheelchair with z50 cannot move.

For uncoupling the motors, the z50 power add-on drive features a brake release mechanism. The brake levers are located at the cover cap of the respective motor.

In the drive mode, the levers at the right and left motor must be in front end position. This is indicated by a symbol of a "wheelchair without attendant".

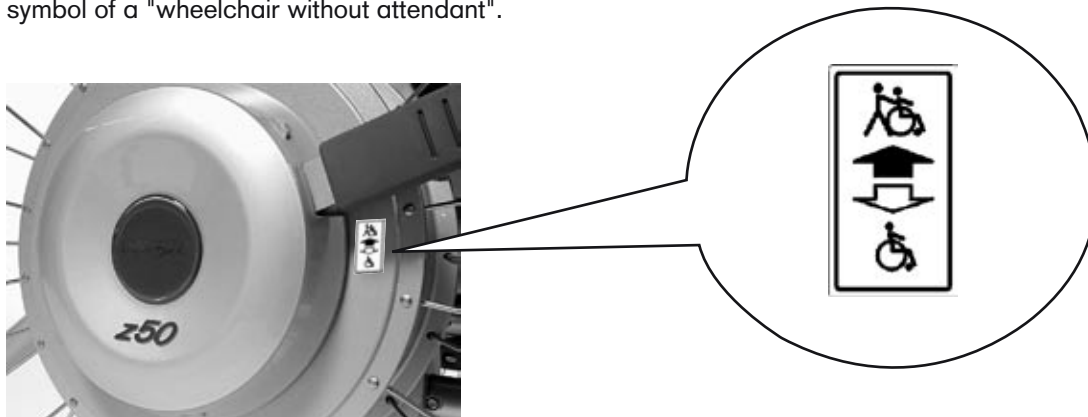
Fig. 10 Brake lever position "drive mode"

With the lever in this position, the motor brake is active as long as the joystick is not deflected.

7.3.1 Push Mode

DANGER

In the push mode, the brake systems of the power add-on drive are inactive. The wheelchair can roll away.

Use the wheel locks when the wheelchair is in the push mode.

CAUTION

If the control shows no error signal when the brake is disengaged, the brake setting must be checked by the specialist dealer.

To release the brake pull the brake lever up. Now the wheelchair is in the push mode. This is indicated by the symbol "wheelchair with attendant".

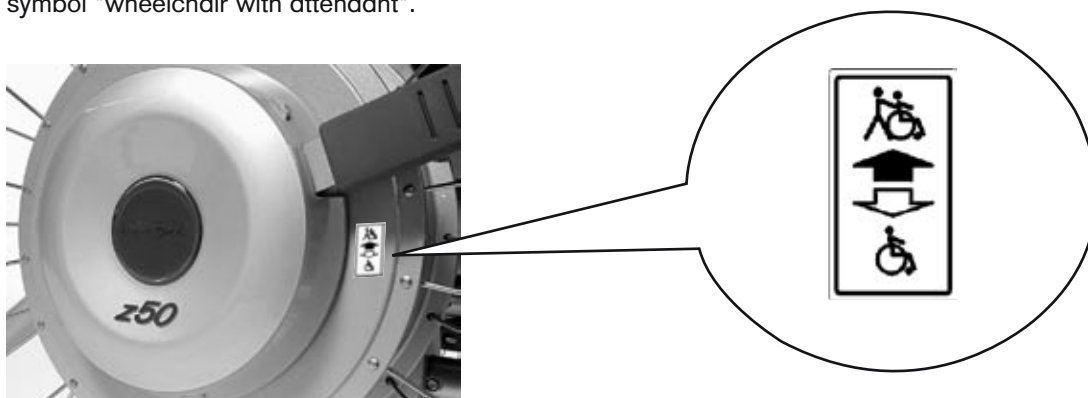
Fig. 11 Brake lever position "push mode"

With the lever in this position, the z50 wheel can rotate freely and has no brake force. To be able to push the wheelchair, the brake levers on the right and left z50 wheel must be in the upper end position. The electric drive mode is not possible with the lever in this position.

When the wheelchair control is switched on, a flashing code (refer to section 9 "Status and Error Analysis") indicates the user that the brake is released.

If the brake levers on the right and left motor are in drive mode position again, the control must be switched off and on for reactivating the driving function.

7.4 Drive-away Lock

The control unit of the z50 power add-on drive features an electronic drive-away lock. For engaging this function please proceed as follows:

1. With the control unit switched on press and hold the On/Off switch.
2. As soon as you hear a signal sound (approx. 1 sec.) release the On/Off switch.
3. Move the joystick to the front until a further signal sound is heard.
4. Now move the joystick to the rear until a further signal sound is heard.
5. A long signal sounds to confirm that the driving function is now blocked, and the wheelchair control is switched off automatically.

For reactivating the driving function please proceed as follows:

1. After switching on, the battery capacity indicator is dark and in the speed level indicator the LEDs continually light up one after another.
2. Move the joystick to the front until a signal sound is heard.
3. Now move the joystick to the rear until a further signal sound is heard.
4. A long signal sounds to confirm unblocking of the driving function, and the battery capacity indicator is lit.

7.5 Battery Indicator

The battery indicator displays different functions:

- Readiness for operation of the control unit
- Remaining battery capacity
- Status and error messages

When in driving mode, the indicator is lit on a steady basis and shows the remaining battery capacity. The indicator consists of 10 LEDs, each corresponding to a value of 10 % of the total capacity. This results in the following information regarding correct use of the battery indicator:

Battery is charged

Charge the battery, if possible

Battery charging is urgently required

Immediately after switching on, the LEDs show the battery capacity status saved before the wheelchair was switched off the last time. The exact state of the remaining capacity will be displayed after driving for a short time.

If only 2 LEDs of the capacity indicator are lit, there is only a little battery capacity remaining. This is indicated by optical and acoustic warnings. If you continue to drive with the power add-on drive, you will run the risk of total battery discharge which would result in battery damage.

NOTE

The warranty becomes void in the case of damages resulting from a completely discharged battery. When the z50 is not used over extended periods of time it must be charged once a week. The battery charger provided by Otto Bock may be exclusively used for charging.


7.6 Battery Charging

The remaining battery capacity determines the distance range of the z50 power add-on drive. Many factors have an influence on the battery's capacity. Besides the ambient temperature, the age of the battery and the time and extent of driving, the way you charge the battery has considerable effects on the capacity and thus on the distance range.

Driving over a longer period of time with only the red LEDs of the battery indicator lit will result in battery discharge and thus battery damage. In addition, there is the risk that the wheelchair with z50 may stop due to zero battery capacity and bring the user into a dangerous situation.

If the z50 is not used for a longer period of time, a charging cycle should be carried out once a week to maintain the battery's capacity.

The indications on the type plate of the battery charger must be identical to the voltage of the mains supply of your country. Charging starts automatically. The current charging state is indicated by the LEDs of the battery charger.

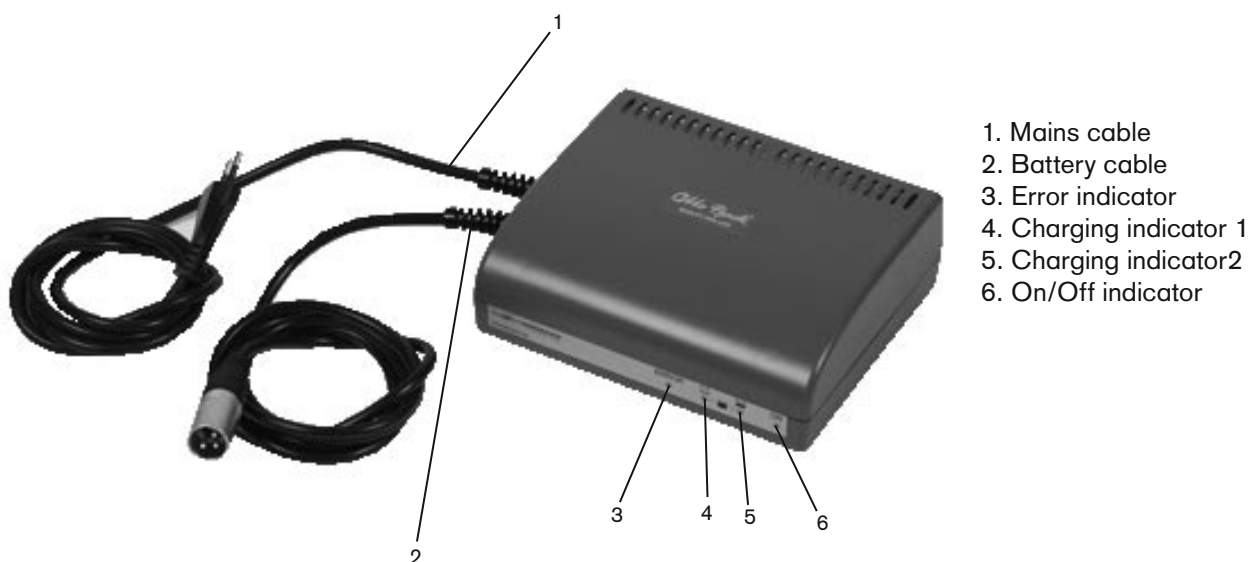- 1. Mains cable
- 2. Battery cable
- 3. Error indicator
- 4. Charging indicator 1
- 5. Charging indicator 2
- 6. On/Off indicator

Fig. 12 Battery charger

7.6.1 Charging Process

The mains plug functions as main switch. As soon as the battery charger is connected to the mains supply the ON lamp is lit. The order for the connection of mains cable and charging cable is irrelevant. The actual charging process starts as soon as the charging plug is connected to the charging receptacle on the control panel. The charging process is indicated by the charging indicator 1. When the charging indicator 1 goes out and the charging indicator 2 is lit, a battery capacity of 80% is reached. When both charging indicators, 1 and 2, are lit, this indicates that the battery is completely charged.

ON lamp	Charging indicator 1	Charging indicator 2	Error indicator lamp	Explanation
On	Off	Off	Off	Mains plug connected / charging plug not connected
On	On	Off	Off	Charging process 0-80%
On	Off	On	Off	Charging process 80-100%
On	On	On	Off	Battery capacity 100%, trickle charge

NOTE

During battery charging, the wheelchair control must be switched off to allow the complete charging current to be fed into the battery.

7.6.2 Error Indicator of the Battery Charger

DANGER

Explosive gases can develop while the batteries are charging. For this reason, provide for sufficient ventilation when using the battery charger in closed rooms. Do not smoke near the battery charger or while handling the batteries. Avoid fire and sparks.

CAUTION

Overheating will destroy the battery charger. When charging batteries, make sure that the cooling ribs on the back side of the connected battery charger are not covered.

ON lamp	Charging indicator 1	Charging indicator 2	Error indicator lamp	Error
On	On	Off	On	1
On	Off	On	On	2
On	On	On	On	3
On	Off	Off	Off	4
On	Off	Off	On	5
Off	Off	Off	Off	6
Off	Off	Off	Flashes	7

Error 1 - 3: The battery charger is overheated because the ambient temperature is too high or because the cooling system does not function properly.

Solution: Protect the battery charger from direct sunlight, do not cover it with any objects or use it in a place with lower ambient temperature.

Error 4: Battery connection is missing. The battery is unconnected, the connector plug is dirty or the battery cable connection is interrupted.

Solution: Check the plug, battery cable and fuse in the battery case.

Error 5: Wrong pole connection of the battery, program error and/or battery fault.

Solution: Please contact your authorized dealer.

Error 6: The battery charger is not connected to the mains supply.

Solution: Check the mains cable and socket.

Error 7: Battery fault

Solution: Please contact your authorized dealer.

The battery charger features a programmed recharging phase. Once the battery is completely charged, the battery charger can remain connected with no risk of overcharging or damaging the battery. Switching the wheelchair control off and on again will make the z50 ready for driving.

The battery charger should be used within the indicated ranges of temperature and humidity only. Make sure that the rubber feet of the battery charger stand on level ground.

When the battery charger is in use, it must be protected from direct sunlight, since this would additionally heat up the charger. Avoid dust and dirt from affecting the function of the battery charger. To clean the charger, use a dry piece of cloth.

7.7 Battery

The standard version of the z50 power add-on drive includes two 12 V lead gel batteries with a capacity of 12 Ah. The batteries are contained in a battery case, which can be removed for transporting or folding the wheelchair.

7.7.1 Removing the Battery

CAUTION

To avoid any damages to the battery case, the battery should be set down appropriately.

The battery case is attached to the wheelchair frame with a special bag. Depending on the way the bag has been mounted it is accessible from the front or from the rear.

To remove the battery, first open the wide hook and loop flap. Now the battery case can be tilted to the front and removed.

Fig. 13 Battery bag

Fig. 14 Removing the battery

To pull off the battery cable, slide the silver locking pin slightly to the rear and turn the plug to the left up to the stop. Now you can pull off the plug, and the battery is disconnected from the wheelchair.

Fig. 15 Pulling off the battery cable

7.7.2 Installing the Battery

DANGER

In case of mechanical damage to the battery case and at the same time to the battery housing, there is the risk of battery leakage. Therefore, please check the battery case for damages before installing the battery.

DANGER

Leaking battery liquid is very caustic. Therefore, protective gloves and safety glasses should always be worn when handling defective gel batteries.

DANGER

If acid splashes into your eyes, immediately rinse them with clear water for a few minutes. Then immediately consult a doctor.
Splashes of acid on the skin or on clothes should be neutralized immediately with acid converter or soap and be rinsed with water afterwards. In case of oral intake of acid, immediately consult a doctor.

To install the battery case, first attach the battery cable plug into the battery case's receptacle and turn it to the right. When the right stop is reached, the safety pin engages and the battery is correctly connected to the control unit. Put the battery case into the bag and close the hook and loop flap.

7.8 Fuse

The 40 A fuse is located in the battery case. After opening the battery cover the fuse holder is freely accessible.

NOTE

During any maintenance work where the battery cover is open, the fuse must always be removed.

7.9 Removing the Control Panel

CAUTION

To avoid any uncontrolled moving of the wheelchair, the control must always be switched off before removing the control panel.

The control panel, which contains the complete electronic control system, can be removed from the wheelchair. To do this, pull the control panel directly to the front. As soon as you have overcome the ratchet point in the mounting rail, the control panel can easily be removed. For transport, the control panel can be put on the seat surface.

Fig. 16 Removing the control panel

Fig. 17 Pulling off the central plug contact

The cable is attached to the wheelchair with hook and loop straps and is connected to the motors and the battery via the central plug contact. The central plug is attached to the crossbrace and, depending on the way it has been mounted, is accessible from the front or from the rear. To completely remove the control panel from the wheelchair, the hook and loop connections must be opened and the central plug be pulled off.

7.10 Anti-tippers

DANGER

Because of the increased risk of tipping due to the motors' drive moment, the z50 may only be used in combination with a manual wheelchair, when corresponding anti-tippers are mounted to the wheelchair.

For the correct setting of the anti-tippers, please refer to the Instructions for Use of the manual wheelchair.

For increased stability against tipping of the z50 power add-on drive in combination with a manual wheelchair, the swinging anti-tippers can optionally be mounted (for more information please refer to section 10.2 Swinging Anti-tippers).

8 Transport and Storage

DANGER

The z50 power add-on drive in combination with a manual wheelchair is **not** suitable for use as a seat for transportation in motor vehicles.

Wheelchair users must **always** transfer to the seats installed in the motor vehicle with the corresponding vehicle restraint systems!

Non-observance of these instructions could lead to injury of the user, as well as of all other passengers in the motor vehicle, in case of an accident.

You can get information on the present state of our measures from your specialized dealer or on the Otto Bock Homepage www.ottobock.de.

To transport the z50 power add-on drive, the system can be disassembled into its individual components. First take the battery out of the bag as described in section 7.8.1 "Removing the Battery". The z50 wheels can be removed as described in section 7.1.3 "Disassembly of the z50 Wheel" and can be loaded separately. The control panel can be removed according to the instructions described in section 7.9 "Removing the Control Panel". Now the wheelchair can be folded as described in the Instructions for Use of the manual wheelchair.

Fig. 18 z50 disassembled

9 Status and Error Analysis

DANGER

If the power add-on drive shows error messages that are not listed or if the wheelchair shows abnormal driving behavior, please contact your authorized dealer immediately.

The following table lists the different error and status messages of the battery indicator and describes the corresponding, possible causes and solutions. This offers the possibility for yourself to identify and eliminate errors of the z50 power add-on drive. If you do not manage to solve the problem, please contact your dealer.

Indicator	Cause	Possible solution
Battery indicator flashes slowly	Battery of the z50 is empty	Charge as soon as possible
More and more LEDs of the battery indicator begin to flash	Battery charger is connected, z50 not ready for driving	Disconnect the battery charger, switch off and on
Battery indicator flashes once every 2.5 seconds	Sleeping mode after longer time of standstill with the wheelchair control switched on	Switch off and on
Battery indicator flashes quickly	Safety-relevant problem, driving function switched off	Switch off and on, check plug connections, battery voltage all right?
Fast flashing of LED 1 (red)	Battery of the z50 empty or poor connection to the battery	Charge the battery or check the plug connections
Fast flashing of LEDs 1-2	Poor connection to the left motor	Check the motor plug connection
Fast flashing of LEDs 1-3	Short circuit on the left motor	Check the motor plug connection
Fast flashing of LEDs 1-4	Poor connection to the right motor	Check the motor plug connection
Indicator	Cause	Possible solution
Fast flashing of LEDs 1-5	Short circuit on the right motor	Check the motor plug connection
Fast flashing of LEDs 1-6	Influence by an external signal (e.g. charging plug)	Remove the charging plug
Fast flashing of LEDs 1-7	Joystick error	Joystick must be in mid-position when switching on the control
Fast flashing of LEDs 1-8	Error of control unit	Check the plug connection
Fast flashing of LEDs 1-9	Brake has been released	Lock the brake or check the motor plug connection
Fast flashing of LEDs 1-10	Overvoltage on the control unit	Check the battery plug connection

10 Options

10.1 Control Panel, Swing-away

NOTE

The driving function is only possible when the control panel is in engaged position.

To allow the user to drive closer to a table or even below the edge of a table, the control panel can be swung to the side with a special control panel holder. For this purpose, the control panel must be pressed outwards. After you have overcome the anterior ratchet point, the control panel can be swung back up into parallel position to the armrest.

Fig. 19 Control panel, swing-away

10.2 Swinging Anti-tippers

DANGER

When starting or accelerating, there is increased risk of tipping.
Use the z50 power add-on drive exclusively with approved anti-tippers. Mount the corresponding anti-tippers on both sides.

DANGER

Driving up to an obstacle in the wrong way can cause you to fall, resulting in a risk of injury to you.
Always approach obstacles (such as steps or curbs) at a right angle.
Please bear in mind that the anti-tippers restrict the obstacle height that can be cleared.

CAUTION

When using lifting platforms, be sure that the wheelchair with z50 power add-on drive is standing centrally on the lifting platform area and that none of the wheelchair's components, such as the anti-tippers, is protruding into the danger area.
Non-observance of this instruction can cause damage to the wheelchair or to the power add-on drive.

For increased stability against tipping of the z50 power add-on drive in combination with a manual wheelchair, the swinging anti-tippers can optionally be mounted. These are two removable supports, which are mounted to the adapter plates of the z50.

Fig. 20 Swinging anti-tippers

The supporting elements are attached to the wheelchair in a way that they can swing in the driving direction. When contacting an obstacle, the two swinging elements can swerve. If the wheelchair tips, however, the two legs of the swinging anti-tippers will contact the ground and thus ensure safe support of the wheelchair.

The anti-tippers can be adjusted to the position of the z50 wheel. In order to ensure safe function of the swinging anti-tippers, the following minimal and maximal dimensions must be observed (s. fig. 25). For making these safety-critical adaptations, please contact your authorized dealer.

Fig. 21 Setting the swinging anti-tippers

10.2.1 Removing the Swinging Anti-tippers

To remove the swinging anti-tipper, first release the locking pin. To do this, press the safety knob and simultaneously pull the locking pin.

Fig. 22 Releasing the locking pin

Now the anti-tipper can be pulled off downwards.

Fig. 23 Pulling off the swinging anti-tipper

Fig. 24 Inserting the locking pin

NOTE

For safe keeping of the locking pin, it is recommended to insert it into the anti-tipper mounting tube.

10.2.2 Attaching the Swinging Anti-tippers

The two anti-tippers are marked with "L" for left and "R" for right. Mount the anti-tippers in accordance with these marks (in driving direction).

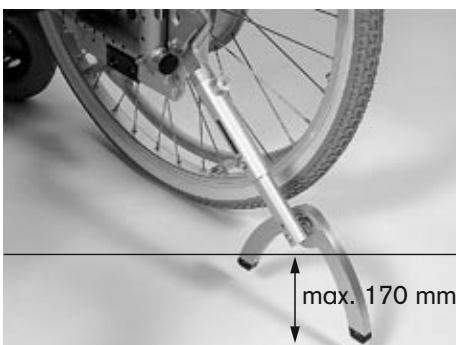
Fig. 25 Mounting position

Fig. 26 Ratchet nose of the swinging anti-tipper

The proper function of the swinging anti-tippers requires correct mounting. The ratchet nose on the anti-tipper mounting tube and the corresponding notch on the anti-tipper tube prevent incorrect mounting. Correct mounting is ensured, when the anti-tipper tube can be pushed over the anti-tipper mounting tube up to stop. Then the ratchet nose is positioned in the notch of the anti-tipper tube. In the end position, the bore hole is free for the locking pin.

Fig. 27 Attaching the locking pin

Completely insert the locking pin into the bore hole until the safety knob springs back into its end position. This is the case, when the locking pin cannot be pulled out without pressing the safety knob.

10.2.3 Joystick Accessories

Fork for tetraplegics, STICK S80, soft ball, golf ball, and flexible control stick shaft

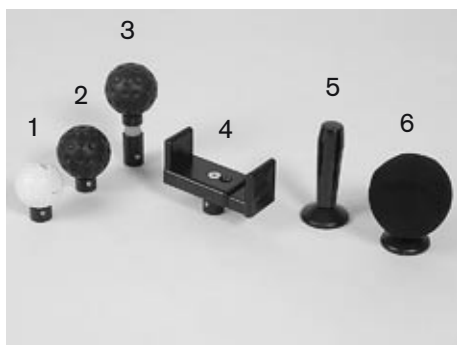
Fig. 28

- 1 Golf ball control stick, small
- 2 Golf ball control stick, large
- 3 Control stick shaft, flexible, incl. golf ball, large
- 4 Fork for tetraplegics, horizontal, for control stick
- 5 Stick S80 (special joystick handle 80 mm)
- 6 Soft ball (black covered)

11 Maintenance and Cleaning

11.1 Maintenance

The correct function of the z50 power add-on drive should be checked before every use. The checks listed in the following table must be carried out by the user at the indicated intervals. Failure to carry out these simple checks may lead to problems arising that could invalidate the warranty.

To be checked	Daily	Weekly	Monthly	Yearly
Tires				
Tire air pressure (printed on the sidewall of the tire)			X	X
Condition of tires			X	X
Batteries				
Battery capacity	X			X

To be checked	Daily	Weekly	Monthly	Yearly
Condition of battery case			X	X
Electronic system				
Control without error message?	>> each time before driving <<			
Does the battery charger show any error messages on the LEDs?		X		X
Driving security				X

If you have any problems carrying out this maintenance, please contact your authorized dealer (refer to section "Service"). Tire change or tire repair will also be carried out by the authorized dealer.

11.2 Cleaning

CAUTION

Water will destroy the drive unit.

Never use a water hose or high-pressure cleaning apparatus for cleaning the power add-on drive.

Water could enter the drive unit and irreparably damage the electronic components.

When cleaning the z50 with water, please be particularly careful with the electrical components.

Please observe the following instructions:

- For cleaning the control panel, battery charger and armrest use only a damp piece of cloth and a mild cleansing solution.
- For cleaning the back upholstery and seat cushion use a dry brush.
- For cleaning wheels and frame use a damp plastic brush.
- Please prevent the electronics, motors and batteries from direct contact with water.
- Do not use any scouring agents.
- Never use a water hose or high-pressure cleaning apparatus for cleaning the wheelchair.

12 Disposal

Normally, the z50 power add-on drive is the property of the health insurance company. Therefore, it is returned to the health insurance company after use. If the wheelchair is privately owned, the following disposal instructions apply:

Please return defective batteries to your dealer when buying new ones.

If the z50 power add-on drive is no longer in use, please give the batteries to a corresponding recycling office. The same applies to wheels, frame, and seat.

The electronic components and motors will be disposed of by the division of your local waste disposal organization which is responsible for electronic scrap.

These guidelines for disposal apply to the manufacturing country of the z50. For use and disposal of the wheelchair in other countries, the disposal regulations of the respective country shall apply.

13 Technical Data of the z50

13.1 Weight Indications

Z50 wheel:	approx. 9 kg
Battery, complete:	approx. 8 kg
Control unit:	approx. 1.2 kg
Maximum load capacity:	125 kg (If your manual wheelchair has a lower maximum load capacity, it must not be exceeded!)

13.2 Electrical Installation

Operating voltage:	24 V
Batteries:	Gel batteries 2 x 12 V 12 Ah (C20h) Optional 2 x 12 V 16 Ah (C20)
Electronic system:	Model: P&G VSI 24 V DC max. 50 A
Motor power:	150 W
Maximum torque moment:	90 Nm
Nominal RPM:	57 1/min

13.3 Battery Charger

Type:	Powercharge Standard 5 A (Automatic battery charger with computer-controlled characteristic curve)
Power requirements:	230 V / 50 Hz
Power:	308 VA
Charging voltage:	8-32.5 V (nominal 24 VDC)
Charging current:	max. 5 A
Weight:	approx. 1 kg
Dimensions:	175 x 140 x 59 mm
Safety class:	II (protective insulation)
Protection class:	IP54

13.4 Driving Data

Speed:	6 km/h
Climbing ability:	10 %
Obstacle height that can be cleared:	1.5 cm
Distance range:	12 km
Turning radius:	Depend on the dimensions of the manual wheelchair

The manufacturer reserves the right to technical alterations!

14 Terms of Extended Warranty

14.1 Subject of the Extended Warranty

- This extended warranty applies to the z50 power add-on drive.
- The extended warranty comprises claims of the medical supply store / service company against Otto Bock HealthCare GmbH and does not affect the statutory warranty claims of the ultimate consumer towards the medical supply store or any other service company being responsible for the fitting of the patient.

14.2 Scope of the Extended Warranty

- Otto Bock HealthCare guarantees, subject to the adherence to the conditions described in paragraph 3 and subject to the observance of the exclusions according to paragraph 4, that beyond the statutory warranty period, up to 4 years after the first use no construction faults, fabrication faults or material faults will occur that considerably impair the wheelchair's usability.
- If 2 years after the first use construction faults, fabrication faults or material faults occur, Otto Bock HealthCare has the choice either to replace the entire z50 power add-on drive or to replace or repair defective parts. This extended warranty does not entitle the medical supply store / service company to further claims for reduction, cancellation of the contract or damage.

14.3 Conditions for Laying Claims

The medical supply store / service company can lay claims under the extended warranty, only if

- it refers to first use,
- the z50 power add-on drive has been employed by a healthcare provider authorized by the health-insurance system,
- the healthcare provider authorized by the health-insurance system passes the complete fitting on along with a report of defects,
- for the construction of custom fabrications made from products fabricated in series production to adapt them to the individual conditions of a patient exclusively medical products with CE-mark are used and these products are used in conformity with the functional purpose,
- modifications on a wheelchair do not go beyond the adaptation intended by Otto Bock HealthCare (e.g. adjusting the legrest length) and the use of the accessories recommended by Otto Bock HealthCare (modular system),
- the one who repairs (adapts) medical products uses exclusively spare parts and add-on parts authorized by the manufacturer (original spare parts) and works according to the provisions given by the manufacturer (see instructions for use).

14.4 Exclusion of the Extended Warranty

The extended warranty is not effective, if Otto Bock HealthCare proves that

- the conditions for laying claims according to paragraph 3 are not complied with,
- the diminution of the wheelchair's usability is due to unprofessional employment of the wheelchair especially retrofits which do not conform to the manufacturer's description,
- the diminution of the wheelchair's usability is due to normal wear and tear especially of the batteries whose usability is usually limited to six months,
- the diminution of the wheelchair's usability is due to incorrect storage, transport or inappropriate as well as unprofessional use and storage,
- the diminution of the wheelchair's usability is due to a change of the patient's physical constitution, e.g. considerable increase in weight,
- the diminution of the wheelchair's usability is due to force majeure.

14.5 Liability

For the undertaking of performance under this extended warranty, Otto Bock HealthCare is liable only for slight negligence in case unessential obligations have been violated; Otto Bock HealthCare assumes no liability for intentional acts of persons performing extended warranty service on our behalf. In addition, damages claims are restricted to the damage foreseeable at the beginning of the undertaking of performance.

14.6 Incidental Provisions

- We are allowed to destroy replaced parts three weeks after return unless the patient or his/her paying party (health insurance company) expressly object.
- Place of fulfillment of performance under this extended warranty is Duderstadt.
- This extended warranty is governed by German civil and commercial law; place of jurisdiction is Duderstadt.

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Otto Bock HealthCare GmbH

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Otto Bock HealthCare GmbH has been certified by the German Society for the Certification of Quality Assurance Systems (DQS) in accordance with DIN EN ISO 9001 standard, reg. no. 779 (management system)